



Bed Handles, Inc. Security-Strap Pictorial Instructions.

For questions or for any problems installing our security straps please call us during regular business hours from 8:30 am – 4:30 pm Central time zone, Monday through Friday, except holidays. The call is toll free at (800)725-6903. You may also write us at: Bed Handles, Inc., 2905 SW 19th Street, Blue Springs, Mo 64015

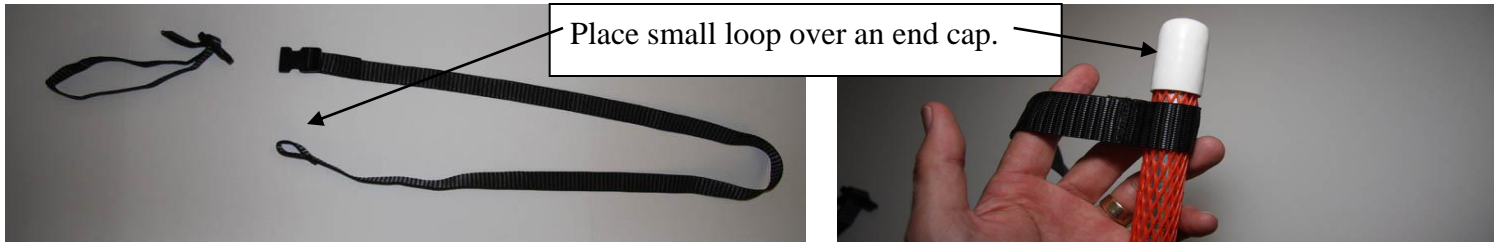

For clarity a single strap is shown. The buckle is separated. There is one long and one short strap. The short strap fits around the bed frame. The long strap attaches to a bed handle leg by a small loop over an end cap.

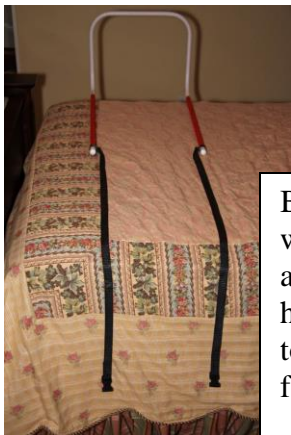


1
Bed Handle shown with both long straps attached. The bed handle is shown on top of the mattress for clarity.

2

For easy installation:

- 1) Slide the bed handle underneath the mattress.
- 2) Hold the ends of the straps and guide them along the foot (or head) of the mattress until they are on the opposite side.

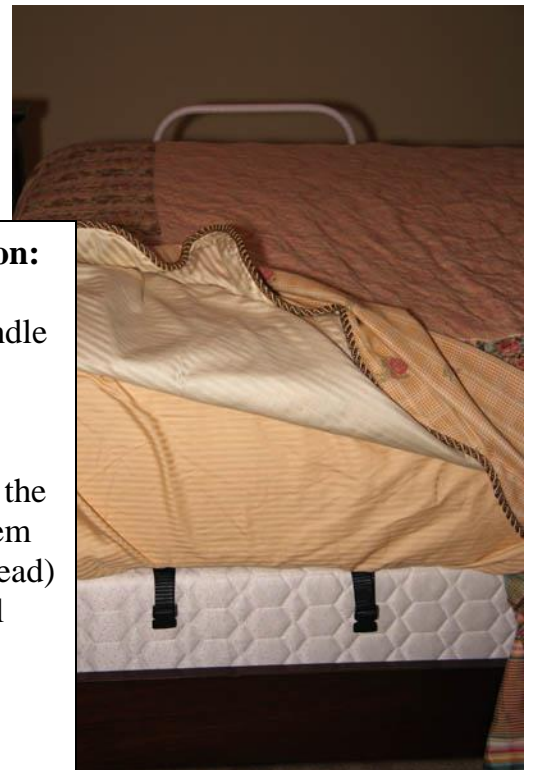

3
Remove the buckle end from the short strap.

Place the strap around your bed frame.



NOTE: There is a top and bottom to a buckle. Notice that the ribs on the center bar of the buckle are on top. If the buckle is assembled while upside down the strap will slip. When properly assembled the strap will **not** slip! Test to be sure.



5
Bring both ends of the strap together.



6
Slide both ends through the slot as shown.



7
Slide both ends through the other slot as shown.



8
Follow the buckle assembly shown above and attach a buckle to the loops around the bed frame.



9
The buckles are easily reached and released whenever the bed handle needs to be moved.

Connect the buckles together. Pull the straps tight by adjusting the buckles.